

Joins

Medstrat is proud to offer a templating service in the cloud designed specifically for orthopedic implant distributors and sales representatives. If you are interested in using our service, please [contact us](#) and we would be happy to explain the benefits of using Joins and help you complete a subscription application form.

Overview

Due to technology shifts and cost containment measures, an increased number of hospitals and orthopedic practices are utilizing digital X-ray (filmless) systems which do not allow you to template with the traditional acetate implant templates. Using our Joins templating software and template packages, you will be able to digitally template, size, and prepare for your cases from anywhere. In addition, with accurate sizing determined pre-operatively, you will be able to bring fewer implants and tools to the OR thus reducing your inventory and lowering sterilization and consignment costs for locations that work with you.

Subscriptions

We currently offer two (2) different subscription types — Single User and Distributor. The Single User Subscription is intended for use by one user with a light case load. A Distributor Subscription is intended for use by multiple users in a high-volume distributor setting that requires [Joins Relays](#) to handle the frequent image transfers.

	Single User	Distributor
Software Installs	3	150
Data Backup	6 years	6 years
Joints To Go	✓	✓
Joints Relay		✓
Admin User		✓
Create Users		✓

How It Works

To begin templating a case under a Distributor Subscription, all you need to do is one (1) of the following:

- (a) [Setup a Joints Relay](#) from the hospital / orthopedic practice to your Joints Server
- (b) Obtain the X-rays as DICOM image files on a CD and import them using the Joints Browser
- (c) Export the DICOM files from their PACS, import them into Joints To Go, and Sync once back at your office

To begin templating a case under a Single User Subscription, you may do so via option (b) or (c). Alternatively, you may upgrade to a Distributor Subscription if you want to use option (a).

Once the DICOM images are imported or [relayed into Joints](#) they can be accessed through the [Joints Browser](#) from anywhere. Just [download](#) and install the Joints Browser on your Mac or PC and you are good to go.

